


The Scout foot by College Park incorporates consistent energy return and adaptable growth plates which allows the foot to accommodate for growth. Using College Park's Intelliweave® technology alongside a durable carbon fibre material, this paediatric foot provides strength, smooth motion and flexibility. This maintenance free foot is available in four different mounting options to meet specific needs, and is lightweight and fresh water friendly.

Waterproof* (fresh water only)

*Waterproof note: Following exposure to chlorinated water, rinse immediately with fresh water and dry.

Features and Benefits

- Accommodates limb growth with the use of stackable growth plates and size up foot shells
- Hybrid material for ultimate strength and flexibility
- Four different mounting options
- Effective energy return
- Lightweight
- Waterproof in fresh water*

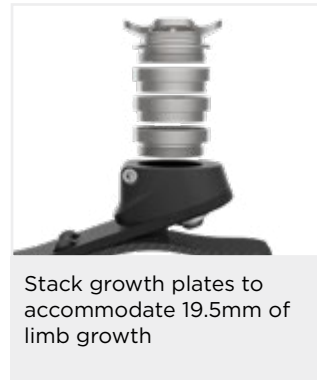

Stack growth plates to accommodate 19.5mm of limb growth



Unique size-up foot shell that allows the user to adjust to a larger size without needing a new foot



Ordering Information

Trial Period - This foot has a 30 day trial period.

Impact Level Descriptions

Low : Daily activities include mostly level ground walking, moving around the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Ordering Example - NSC-LCM-17-D4

Part ID	Side	Shell Colour	Mounting	Size	Firmness Category
NSC	L		M		D
NSC	R		M		D

Caucasian C
Tan T
Brown B

13-22cm

1-4



Scout Adapters & Receivers		
Part Number	Description	
1 NTA-SP	Threaded Pyramid Adapter	
2 NTR-SP	Threaded Pyramid Receiver	
3 NTXA-SP	Threaded Exo Adapter*	
4 NTLAS-SP	Threaded Lamination Adapter (S)	
5 NTLAM-SP	Threaded Lamination Adapter (M)	
6 NTLAL-SP	Threaded Lamination Adapter (L)	

*(only available for sizes 16cm and above)

Firmness Category			
Weight (kg)	13-15cm	16-18cm	19-22cm
0-25	1	1	1
26-33	-	2	2
34-45	-	3	3
46-60	-	-	4

Spare Parts Ordering Information


Foot Shell	Size (cm)
NSC-S ■●-▲	13-22

■ = "L" or "R" for Left or Right
● = Colour (C = Caucasian, T = Tan, B = Brown)
▲ = Foot size in cm (13-22)

CPI Spectra Sock	Size (cm)
NTP-SSX-1618	13-18
NV-SS-2124	19-22

Product Specification

Sizes	Weight Limit	Heel Height	Build Height	Weight of Unit
13-15cm	25kg	5.8mm	See Next Page	183-360g
16-18cm	45kg	6.4mm		
19-22cm	60kg	9.5mm		
Impact Level	Warranty	Foot Shell		
	1 year foot shell - 6 months	Caucasian, Tan or Brown Standard		

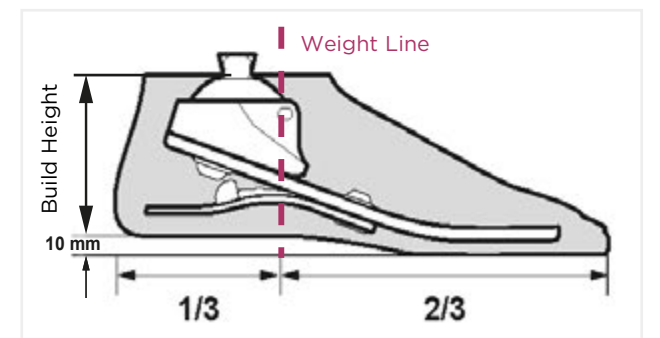















Product Specification - Continued

Build Height				
Size	Lamination Adapter	Pyramid	Receiver	Exo Block
13-15cm	4.7cm	4.6cm	5.3cm	-
16-18cm	5.1cm	4.9cm	5.7cm	10.4 cm
19-22cm	5.4cm	5.3cm	6.0cm	10.8 cm

Spare Parts Ordering Information

Scout Adapters

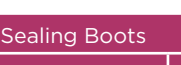	Part Number	Description
	NTFH-800	Foot Horn, Stainless Steel
	NSC-LA-FAB	Lamination Adapter Fabrication Kit
	NCP-TCS-P	Tube Clamp Screw, M4 x 14mm, Pediatric
	NTP-SH-8X16	Exo Mounting Bolt, M8 x 16mm
	NTP-SH-8X30	Exo Mounting Bolt, M8 x 30mm
	NSC-SH-8X18	Mounting Bolt, M8 X 18
	NSC-SH-8X22	Mounting Bolt, M8 X 22
	NSC-SH-8X25	Mounting Bolt, M8 X 25
	NSC-SH-8X10	Mounting Bolt, M8X 10
	NTP-RBK	Pyramid Tool Retaining Bolt Kit (4pc) M5 x 10mm
	NSC-HWK-1315	Scout Wedge Kit, 13-15cm
	NSC-HWK-1618	Scout Wedge Kit, 16-18cm
	NSC-HWK-1922	Scout Wedge Kit, 19-22cm
	NCA-409	Cyanoacrylate Adhesive, 409 Gel (CA glue)

Sealing Boots

	Part Number	Description
	NTP-SB ■● -1618	Sealing boot, (left or right), 16-18cm
	NTP-SB ■● -1921	Sealing boot, (left or right), 19-21cm




■ = "L" or "R" for Left or Right

● = Colour (C = Caucasian, T = Tan, B = Brown)










Finishing Rings

	Part Number	Description
	NTP-FR ■● -1618	Sealing boot, (left or right), 16-18cm
	NTP-FR ■● -1921	Sealing boot, (left or right), 19-21cm

Exo Blocks and Lamination Mounting Adapters

	Part Number	Description
	NSC-GK ■C-1618	Alignable Exo Block Kit (left or right) 16-18cm
	NSC-GK ■B-1922	Alignable Exo Block Kit (left or right) 19-22cm
	NTP-EXPK-22	Exo Pyramid Tool Kit, 22 mm for temporary endoskeletal alignment during test fittings.

Exo Blocks and Lamination Mounting Adapters

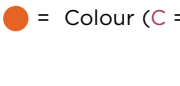	Part Number	Description
	NSC-GK ■C-1618	Growth Plate Kit Caucasian, 16-18cm
	NSC-GK ■B-1618	Growth Plate Kit Brown, 16-18cm
	NSC-GK ■T-1618	Growth Plate Kit Tan, 16-18cm
	NSC-GK ■C-1922	Growth Plate Kit Caucasian, 19-22cm
	NSC-GK ■B-1922	Growth Plate Kit Brown, 16-18cm
	NSC-GK ■T-1922	Growth Plate Kit Tan, 19-22cm
	NSC-LAGK	Scout Lamination Adapter Growth Plate Kit
	NSC-TLGP-1322	Scout Threaded Lamination Growth Plate 13-22cm
	NSC-LGP-1322	Scout Lamination Growth Plate 13-22cm

■ = "L" or "R" for Left or Right

● = Colour (C = Caucasian, T = Tan, B = Brown)

