

LA Liners

A medical-grade mineral oil is contained in all Silipos gels, which gradually diffuses to moisturise and lubricate the skin.

Even in changing environments, the gel has the ability to conform to whatever shape or surface contour it comes into contact with. Because of the elasticity and conformability, the interface can adapt to the patient rather than the other way around.

The soft gel of the LA liner provides superior skin conditioning and comfort, for the low to moderately active patient, and is especially effective when used as a cushion liner with suspension sleeve, with or without a suction valve.

Features and Benefits

- Medium durometer gel makes this liner particularly suitable for low to moderately active patients, whilst providing increased skin care and comfort
- Keeps skin and scar tissue soft and pliable
- Prevents skin from becoming dry and fights the development of calluses
- Helps prevent contact dermatitis and skin sensitivity problems
- Available in tapered and uniform profiles for both pin and cushion liners
- Available in the standard and the more conical, plus version
- 3mm, 6mm and 9mm gel options with 3mm thickness posteriorly to accommodate knee flexion


Ordering Information

LA Pin Liner			
Part Number	Size	Thickness	Shape
N18101	Medium	3mm	Uniform
N18102	Medium	3mm	Tapered
N18103	Medium	6mm	Uniform
N18104	Medium	6mm	Tapered
N18105	Medium	9mm	Uniform
N18106	Medium	9mm	Tapered
N18107	Medium +	3mm	Uniform
N18108	Medium +	3mm	Tapered
N18109	Medium +	6mm	Uniform
N18110	Medium +	6mm	Tapered
N18111	Medium +	9mm	Uniform
N18112	Medium +	9mm	Tapered
N18113	Large	3mm	Uniform
N18114	Large	6mm	Uniform
N18115	Large	9mm	Uniform
N18116	Large +	3mm	Uniform
N18117	Large +	6mm	Uniform
N18118	Large +	9mm	Uniform
N18119	X-Large	3mm	Uniform
N18120	X-Large	6mm	Uniform

LA Cushion Liner			
Part Number	Size	Thickness	Shape
N18141	Medium	3mm	Uniform
N18142	Medium	3mm	Tapered
N18143	Medium	6mm	Uniform
N18144	Medium	6mm	Tapered
N18145	Medium	9mm	Uniform
N18146	Medium	9mm	Tapered
N18147	Medium +	3mm	Uniform
N18148	Medium +	3mm	Tapered
N18149	Medium +	6mm	Uniform
N18150	Medium +	6mm	Tapered
N18151	Medium +	9mm	Uniform
N18152	Medium +	9mm	Tapered
N18153	Large	3mm	Uniform
N18154	Large	6mm	Uniform
N18155	Large	9mm	Uniform
N18156	Large +	3mm	Uniform
N18157	Large +	6mm	Uniform
N18158	Large +	9mm	Uniform
N18159	X-Large	3mm	Uniform
N18160	X-Large	6mm	Uniform

Gel Liners - Silipos - LA Liners

Sizing Information

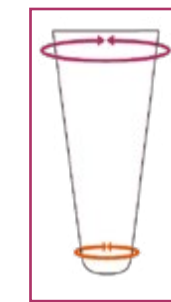

Proximal Circumference
(Measure 38cm from Distal End)

Distal Circumference
(Measure 4cm from Distal End)

Size	Distal Relaxed	Distal Fully Stretched	Proximal Relaxed	Proximal Fully Stretched
Medium	20cm	30cm	23cm	41cm
Medium +	23cm	30cm	33cm	53cm
Large	23cm	38cm	38cm	56cm
Large +	30cm	41cm	41cm	68cm
X-Large	30cm	51cm	41cm	76cm

By measuring your patient, and finding out where their measurements are in relation to the liners measures, you can select a liner which is firm in fit (for more active patients), relaxed in fit (for less active patients who will be sitting more), or for the middle fit, which is suitable for the majority of patients. An example would be if your patient's distal measurement is 28cm, and the proximal measurement is 42cm, then he/she would fit perfectly in the mid-range of the Medium+.

Uniform - has a uniform thickness throughout the liner with the exception of the gel in the posterior.

Tapered - has 3mm additional gel in the distal 10cm for bony and conical residual limbs. The gel's flexibility is similar to having a distal end pad placed in the liner.

Technical Information

Material	Soft Polymer Gel
Matrix	None
Length	Approx 38cm
Amputation Level	Transtibial
Distal Connector	Pin Adapter or Cushion
Impact Level	Low/Medium (K1 to K2)
Warranty	6 months

